




# DENTAL R.A.T.<sup>SM</sup>

## Hands-Free Perio Charting

### Dental R.A.T. 2.0- Wireless Installation


- 1 Plug the USB Dongle into the computers USB port. The Dental R.A.T. 2.0 will automatically upload itself to the computer.
- 2 Put (3) AA Batteries into the back of the unit. Reminder: 
- 3 Your Dental R.A.T. is ready to operate! Watch the training video online at [www.dentalrat.com](http://www.dentalrat.com) to learn tips and shortcuts for use with your software.

### How it Works

**Mouse**  
Joystick at the top moves your mouse cursor

**Numbers**  
The four buttons across the center are numbers 1-8. If each button is pushed down for less than one second they are 1, 2, 3, and 4. If each button is pushed down for longer than one second they become 5, 6, 7, and 8.

(for numbers higher than 8 please see software training videos)

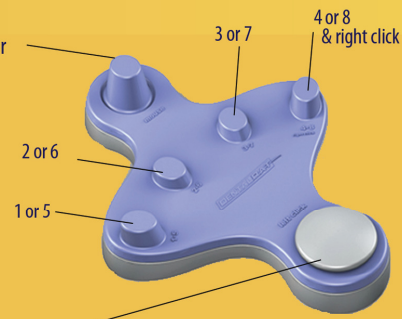

**Left Click**

The bottom button acts as a heel rest but is also the left mouse click. This allows the cursor to click on items just as if you were to left mouse click with a hand operated mouse. To activate the left mouse click "rocking back" by lifting the front of your foot. Take care to NOT press down directly as this could damage the unit.

### Audio Feedback Option

Audio feedback is included at no additional charge, please visit our website where you will find a download to match your system. (PC/Mac)

### Adjusting Mouse Cursor Speed

To speed up- Press and hold numbers 1 and 2 simultaneously on the unit, while holding the numbers press the mouse cursor up until you reach the desired speed.

To slow down- Press and hold numbers 1 and 2 simultaneously on the unit, while holding the numbers pull the mouse cursor toward heel until you reach the desired speed.

### Standard Warranty

Dental R.A.T., LLC warrants that this product shall be free from defects in materials and workmanship for one year. If the product is defective, Dental R.A.T., LLC's sole obligation under this warranty shall be at its election, subject to the terms of this warranty, to repair or replace any parts deemed defective. This warranty is for the exclusive benefit of the original purchaser and is not assignable or transferable. THIS WARRANTY IS MADE IN LIEU OF ALL OTHER WARRANTIES, EXPRESS OR IMPLIED AND ANY IMPLIED WARRANTIES, INCLUDING THE IMPLIED WARRANTIES, INCLUDING THE IMPLIED WARRANTIES OF MERCHANTABILITY AND FITNESS FOR A PARTICULAR PURPOSE, ARE LIMITED TO THE TERMS OF THIS WARRANTY, UNLESS OTHERWISE PROHIBITED BY LAW.

**Exclusions:** To the fullest extent permitted by law, Dental R.A.T., LLC expressly disclaims and excludes from this warranty any liability for damages, including but not limited to general, special, indirect, incidental consequential, aggravated, punitive or exemplary damages, and economic loss (even if Dental R.A.T.™ is informed of their possibility), third party claims against you, include for any loss or damages, and claims or damages related to loss of, or, damage to, your records or data. This warranty will be voided by misuse, improper physical environment, improper repair, act of God, war or terror, and improper service, operation or handling.

**Warranty Return Merchandise:** Any claim under this warranty must be made in writing to Dental R.A.T., LLC or by calling technical support, within 30 days after the owner discovers the circumstances giving rise to any such claim, and must include a copy of warranty document, a receipt or copy of your invoice marked with the date of purchase, and the original packaging. You may obtain service by calling the technician. Package must be mailed to Dental R.A.T.™ postage prepaid.

**Return Policy:** Returned Merchandise other than for warranty is subject to a 25% restocking fee within 30 days of purchase. After 30 days returns will not be accepted.



### Technical Support

We always offer FREE technical support and FREE online training! Dental R.A.T.™ provides Technical Support Monday – Friday from 8:00am – 5:00pm Mountain Standard Time. Our office is closed on Holidays.

Toll Free: (877) 278-0061

Live Chat: [www.dentalrat.com](http://www.dentalrat.com)

Outside US: +1 (208) 489-1590

Email: [support@dentalrat.com](mailto:support@dentalrat.com)

©Beckmer Products, Inc.

Dental R.A.T. is a trademark of Beckmer Products, Inc.

Dental R.A.T.™ is a division of Beckmer Products, Inc.

Patent 7,369,116 / 7,554,526 and U.S. and Foreign Patents Pending.